

APTIMA COLLECTION


Orange Tube	White Tube	
VAGINAL	URETHRAL/ENDOCERVICAL	
• Vaginosis/Vaginitis Plus GC/CT/Mycoplasma (VAGPLS) (2 Kits) • Vaginosis/Vaginitis Plus GC/CT (VAGGC) (2 Kits) • Vaginitis Comprehensive Panel (VAGCOM) • Vaginosis Bacterial, TMA (BVTMA) • Candida Vaginitis, Trichomonas, TMA (CVTTMA) • STI Comprehensive Panel (GCTM) • Mycoplasma Genital RNA, TMA (MYCRNA) • Chlamydia and GC, RNA, TMA (GCCTMA) • Chlamydia, GC, and Trichomonas (GCT) • Chlamydia Trachomatis, RNA, TMA (CTRNAB) • Neisseria Gonorrhea, RNA, TMA (GCRNAB) • Trichomonas Vaginalis by TMA (TRTMAB)	• STI Comprehensive Panel (GCTM) • Mycoplasma Genital RNA, TMA (MYCRNA) • Chlamydia and GC, RNA, TMA (GCCTMA) • Chlamydia, GC, and Trichomonas (GCT) • Chlamydia Trachomatis, RNA, TMA (CTRNAB) • Neisseria Gonorrhea, RNA, TMA (GCRNAB) • Trichomonas Vaginalis by TMA (TRTMAB)	
	Sterile Cup	
	URINE	
	• STI Comprehensive Panel (GCTM) • Mycoplasma Genital RNA, TMA (MYCRNA) • Chlamydia and GC, RNA, TMA (GCCTMA) • Chlamydia, GC, and Trichomonas (GCT) • Chlamydia Trachomatis, RNA, TMA (CTRNAB) • Neisseria Gonorrhea, RNA, TMA (GCRNAB) • Trichomonas Vaginalis by TMA (TRTMAB)	
COLLECTION		
BEFORE YOU GET STARTED		
• DO NOT spill or discard any of the liquid that comes in the collection device ◦ If liquid is spilled, a new kit will need to be collected • DO NOT use scissors to break/cut swab • DO NOT break swab before placing in tube • DO NOT use any other tube or swab		
VAGINAL (PINK SWAB) 1. Open PINK swab without touching any surface 2. Hold swab above the scored/black line 3. Insert swab 5 cm past the vaginal opening and rotate swab 10-30 seconds touching the vaginal walls 4. Remove swab without touching skin 5. Place swab into ORANGE tube without spilling liquid from tube 6. Break swab at the scored/black line and allow swab to fall into tube 7. Tightly close lid on tube 8. Repeat if test requires 2 tubes	MALE URETHRAL (BLUE SWAB) 1. Pull foreskin back if needed 2. Hold BLUE swab above the black line 3. Insert BLUE swab 2-4 cm into the penis and rotate clockwise 2-3 seconds 4. Remove swab without touching skin 5. Place swab into white tube without spilling liquid from tube 6. Break swab at the black line and allow swab to fall into tube 7. Tightly close lid on tube	ENDOCERVICAL (BLUE SWAB) 1. Open white cleaning swab and remove excess mucus from cervical and surrounding mucosa • Discard white swab 2. Hold BLUE swab above the scored/black line 3. Insert BLUE swab into cervical canal and rotate clockwise 10-30 seconds (avoid contact with vaginal mucosa) 4. Remove swab without touching skin 5. Place swab into white tube without spilling liquid from tube 6. Break swab at the scored/black line and allow swab to fall into tube 7. Tightly close lid on tube
URINE (STERILE CUP) 1. Do NOT provide a clean catch urine 2. Collect initial urine stream directly into sterile cup 3. Patient should not urinate 1 hour prior to collection		

APTIMA COLLECTION


Sensitivity by Collection Device and Source

	Vaginal Pink Swab	Endocervical Blue Swab	Male Urethral Blue Swab	Male Urine Cup	Female Urine Cup
Chlamydia Trachomatis	96.6%	94.2%	95.6%	97.9%	86.9%
Neisseria Gonorrhoeae	96.0%	99.2%	99.1%	98.5%	90.7%
Trichomonas Vaginalis	100.0%	100.0%	No Data	100.0%	95.1%


Aptima Collection Devices

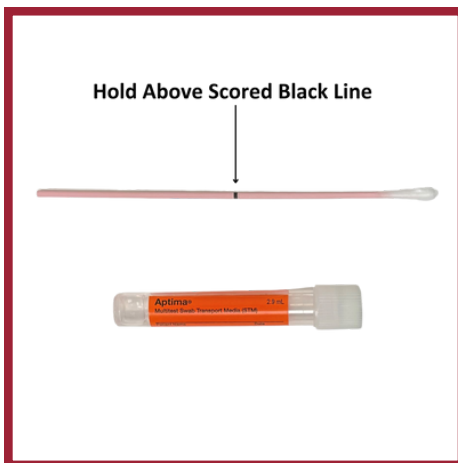

Multisex Swab

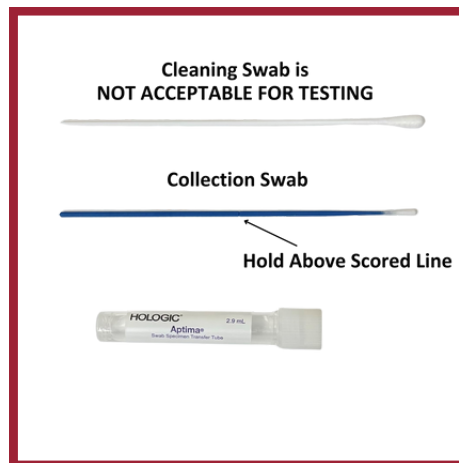

Unisex Swab

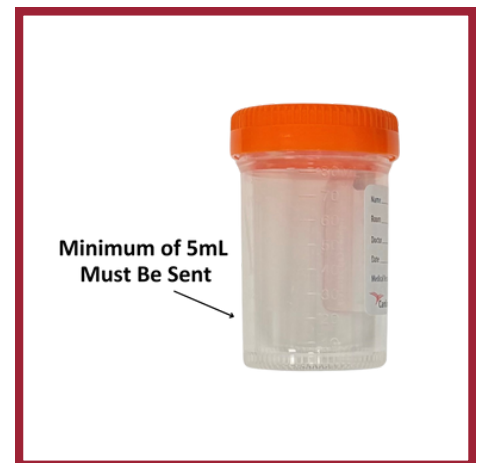

Urine Cup

